

NEW!

PARTIAL MOULDABLE BORDER-LOCK® DISPOSABLE IMPRESSION TRAYS

THE FIRST SEMI CUSTOM TRAY THAT CAN BE ADAPTED BY HAND



1. KEEP THE SELECTED TRAY IMMERSSED IN WATER (70°C) FOR ONE MINUTE
2. ONCE SOFTENED ADAPT THE TRAY BY HAND
3. AFTER ADAPTION THE TRAY WILL ASSUME ITS NORMAL RIGIDITY

- A DEVELOPMENT OF SCHREINEMAKERS IN COOPERATION WITH ABSOLUTE CERAMICS (BIODENTIS GmbH).
- PRECISE CUSTOMISATION BY MEANS OF THERMOPLASTIC MOULDABILITY
- TRAY SIZE NR. 1 FOR I. AND III. QUADRANT IMPRESSIONS
- TRAY SIZE NR. 2 FOR II. AND IV. QUADRANT IMPRESSIONS



- Keep the tray immersed in water (70°C) for one minute
- Adapt the tray by hand
- Keep desired form stable for 5 seconds

- Left image: original tray
- Right image: adapted tray



- Single handed positioning and stabilisation

(Images: absolute Ceramics (biodentis GmbH))

Clinical experience by Dentists:

- “Can be held perfectly”
- “Material saving shape”
- “Increased dynamic pressure”
- “Here somebody has really been thinking”

Partial mouldable Border-Lock® disposable impression trays - Order Options:

Art.Nr. 5480G **Box 10 partial trays**
5x size nr. 1 + 5x size nr. 2

Available from your dental dealer

CLAN 

Clan Dental Products
P.O. Box 2606 - NL-6026 ZG Maarheeze - The Netherlands
Tel +31 495593351 - Fax + 31 495591315
clan@xs4all.nl - www.borderlock.com