

NEW!

MOULDABLE BORDER-LOCK® DISPOSABLE IMPRESSION TRAYS

THE FIRST SEMI CUSTOM TRAY THAT CAN BE ADAPTED BY HAND



1. KEEP THE SELECTED TRAY IMMERSED IN WATER (70°C) FOR ONE MINUTE

2. ONCE SOFTENED ADAPT THE TRAY BY HAND

3. AFTER ADAPTION THE TRAY WILL ASSUME ITS NORMAL RIGIDITY

- AS FROM 1 MAY 2008 THE MOULDABLE DISPOSABLE TRAYS WILL REPLACE THE CURRENT BORDER-LOCK® DISPOSABLE TRAYS
- INDICATED FOR PRECISION IMPRESSIONS WHEN USING THE MONOPHASE OR SANDWICH TECHNIQUE
- CLOSER FIT CREATES OPTIMAL DYNAMIC PRESSURE IN THE IMPRESSION MATERIAL
- BORDERLOCK® IMPRESSION TRAYS ORIGINALLY DEVELOPED BY SCHREINEMAKERS



• Mouldable dentulous tray

- Keep the tray immersed in water (70°C) for one minute
- Use of a hot water bath is ideal (Dux Dental)
- A simple alternative is to use hot water (70°C) in a suitable container



- Adapt the tray by hand
- Keep desired form stable for 5 seconds
- Check the fit in the mouth and if needed, repeat the moulding procedure

Clinical experience by Dentists:

“These mouldable trays are SUPER!! The idea of self-customising the trays is unique”
Dr. van Thoor, Dentist, Germany.

“I am very satisfied with this development, because I have more and more patients with pronounced mandibular bony ridges”, Dr. Nolens, Dentist, Belgium

“The mouldable tray is perfect”, Dr. Brinkman, Dentist, Amsterdam University

“I feel that the trays meet a deficiency in the market place and would be a tool of great benefit, in any Denture Clinic”, Mr. Claassens, Dental Prosthetist, Australia

“The mouldable trays are superb and great advantages for the practice, when are they available?”
Dr. Firla, Dentist, Germany

Mouldable Border-Lock® disposable impression trays - Order Options:

Art.Nr. 5400G **Trial Set 12 trays dentulous / edentulous**
1x dentulous no. 42, 43, 44, 45, 51, 52, 53, 54, 1x edentulous no. 22, 23, 32, 33

Art.Nr. 5401G + 5402G **Intropack 98 trays dentulous**
1x Set 10 upper + 1x Set 8 lower en 8x refill of 10 trays no. 42-45, 51-54

Art.Nr. 6400G **Intropack 40 trays edentulous**
1x Set 5 upper + 5 lower and 30 trays refill van no. 21-25, 31-35

Refill dentulous: 10 trays per size

Refill edentulous: 5 trays per size

Available from your dental dealer

Clan Dental Products

Po.Box 2606 - NL-6026 ZG Maarheeze - The Netherlands

Tel +31 495593351 - Fax + 31 495593351

clan@xs4all.nl - www.borderlock.com

CLAN

